MADE IN GERMANY

OKE MUND-NASEN-MASKE

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MADE IN GERMANY

OKE FACE-MASK

Set with exchangeable protective fleece (one-way), retaining rubbers and reusable multiclips

USER MANUAL

This mask is not certified and provides basic protection. Therefore OKE Automotive GmbH & Co. KG assumes no liability for any damage and does not guarantee any special protection. The mask may only be used in accordance with the applicable health and safety standards and only if the concentration of dirt and hazardous substances does not directly pose a danger to life or health.

Please observe all hygiene rules such as regular hand washing and a distance of at least 1.50 meters if you use it as protection against infection!

OKE Automotive GmbH & Co. KG
Nobelstraße 7 / 48477 Hörstel / Germany / oke-shop.de

The fleece is for single use and must not be worn continuously for more than eight hours. After use, the multiclips must be removed for further use. Thorough disinfection of the clips is necessary before reuse. The fleece must be disposed of.

Unused masks must be transported and stored in such a way that they are not exposed to direct sunlight or frost, do not come into contact with solvents and cannot be damaged by contact with hard surfaces or objects.

In accordance with European regulations for the protection of health at work, the user's attention is drawn to the fact that the mask may cause allergic reactions when in contact with the skin. If this is the case: leave the danger zone, remove the mask and seek medical advice.

Do not use in poorly ventilated areas where there is a lack of oxygen or where there is a risk of explosion. The mask must be correctly fitted and removed to ensure that protection is achieved.

It is unlikely that the tightness requirements can be met if whiskers protrude from under the face seal.

The mask should be disposed of and replaced in the following cases:

1. If the mask is removed in a contaminated area
2. If the mask causes breathing difficulties.
3. If the mask is damaged.

WARNING NOTE

The user must be instructed in the fitting and removal of the mask and must read these instructions before fitting. The instructions and warnings for use must be followed at all times during exposure, otherwise the effectiveness of the mask may be reduced, resulting in illness or long-term health damage.


Before use, check the integrity of the mask and ensure that the mask is suitable for the application.

MATERIAL


Protective fleece: PP spunbond
Multiclip: PP
Retaining rubber: rubber with fabric coating

ASSEMBLY INSTRUCTION

1. The protective fleece are joined in packs of 2 for production purposes. Before building up the mask, separate them from each other!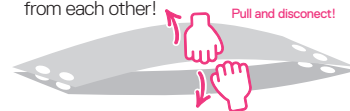

2. If the protective fleece unfolds during separation, return it to its original folding shape.

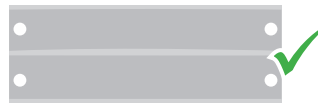3. Place a protective fleece on a smooth surface.

4. Insert the pins of the multiclips through the holes of the protective fleece.

5. Close both multiclips.

6. Knot the retaining rubber at the end.

7. Thread the retaining rubber.


8. Re-knot the retaining rubber, if too long.

9. If necessary, shorten the protruding ends of the retaining rubbers with scissors.

10. Put the mask on with the central slit facing inwards, towards the mouth.

11. Pull the inner part of the mask up over the nose and down over the chin.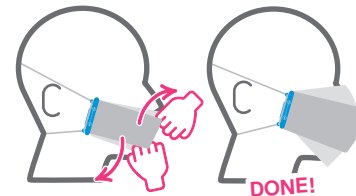

CHANGE FLEECE

1. Open the quick release of the multiclips.
2. Dispose of used fleece.
3. Insert new fleece.
4. Close the quick release of the multiclips.